

Setting Instructions

In the main page “Welcome to use”, press the first key for 5 secs, then turn to the setting page, see the following page



1---time setup ,press the confirmation key to setting time page



up-down to adjust the time ,date, then confirm

2—print record, if you want to print the former record, use this function, press the last key confirmation to choose which record you want to record



Up –down to choose, 1 means the last one record, 2 means the last 2 records, it can make a record with many times.....



3---Ambient pressure—when alarm the pressure is not in safe, press the last key (confirmation key) to reset the local pressure..



4— dry setup, The dry setup range is 20~90, the data is bigger, means the dry temperature is higher

when cycle finished, if you find the instruments is wet, you want to make it dry, 3 ways to make it dry

a...make the front 2 machine feet little higher than the back feet, run cycle to check the dry ,if still wet, so the next step

b---clean the chamber filter, run cycle then after check dry ,still wet, then to the third way , setting dry data




The initial data is 30, then you can change it to 35, press up key adjust, then confirm, if still wet, change to 38...



5---steam heat, when chamber temperature is too low or too high, you can adjust this data

The setting range is 0~20, if the chamber temperature is too low, then turn down the data, make it smaller, if the chamber temperature is too higher ,then turn up the date, make it bigger data..

The initial data is 10

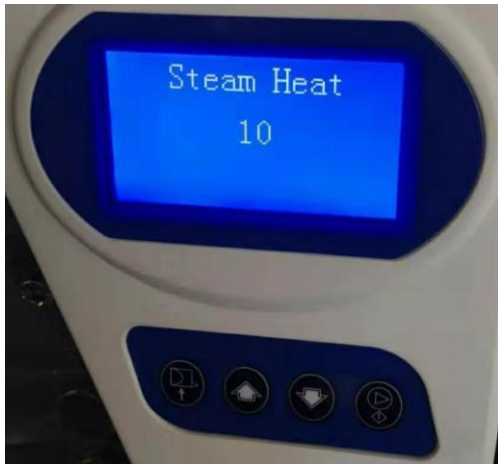

6—machine information , it shows serial number, counter ..



- | | | | | |
|---------------------|------------|-----------|-----------|------------|
| 1. initial language | 8.English | 9.Spanish | 10.Frensh | 11.Italish |
| 12.Germany | 13.Russian | | | |

For languages, we can add more... but all need customers to help translation, because customers know well using with their local languages ...